

Ziru Quick Start

Turn On Ziru

- Press Power button once;

OR, failing that:

- Press and hold Power button for 5 seconds.

Ziru also turns on automatically when you disconnect the power cable.

Power button is located at the bottom right corner of Ziru.

Voice Commands

- Next
- Previous
- Zoom In / Out
- Battery
- Brightness Up/Down
- Volume Up / Down
- Read Text
- Light On / Off
- Reset Zoom
- Voice Feedback On
- Set Slow Speech
- Zoom Four
- Volume Fifty
- Frame Rate Up
- Yellow Outlines
- Black Background

Turn Off Ziru


- Press Power button once.

Charge

- Insert USB cable into charger;
- Plug in Charger into wall socket;
- Insert the other end of the cable into the charging port.

Charging port is located on the left hand side in the centre.

Customer Support Dodrotu Limited

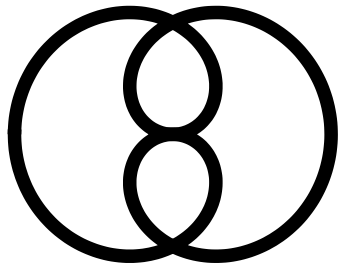

Email:
support@dodrotu.com
Phone:
+44 07577 669 499
Web:
www.dodrotu.com/ziru

About Your Ziru

Version: 4.32.1046
Build date:
2026-04-10
Variant:
Enter Glass Dragon

Health and Safety Warning

This variant of Ziru has a number of limitations.

By using the product you agree to read and abide by the Product Limitations and Health and Safety Warnings sections of Ziru User Guide.

Further Info

Full functionality is described in detail in an online User Guide.

Ziru User Guide

Electronic copy is available online by following this link:

www.dodrotu.com
/ziru/user-guide
/4.32.1046.pdf

