

ZIRU: A Very Quick Start

Turn On

- Press Power button once;

OR, failing that:

- Press and hold Power button for 5 seconds.

Ziru also turns on automatically when you disconnect the power cable.

Power button is located at the bottom right corner of Ziru.

Voice Commands

- Next
- Previous
- Zoom In
- Zoom Out
- Reset Zoom
- Display Battery
- Flash On (Off)
- Brightness Up
- Brightness Down
- Volume Up (Down)
- Yellow Outlines
- Black Background

Turn Off


- Press Power button once.

Charge

- Insert USB cable into charger;
- Plug in Charger into wall socket;
- Attach magnetic end of the cable to the charging port.

Charging port is located on the left hand side in the centre.

Customer Support Dodrotu Limited

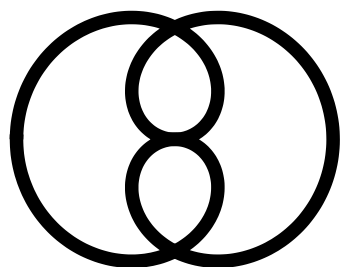

Email:
support@dodrotu.com

Phone:
+44 07577 669 499

Web:
www.dodrotu.com/ziru

About Your Ziru

Version: 4.4.822

Build date:
2024-01-26

Variant:
Enter King Dragon

Health and Safety Warning

This variant of Ziru has a number of limitations.

By using the product you agree to read and abide by the Product Limitations and Health and Safety Warnings sections of Ziru User Guide.

Further Info

Full functionality is described in a gripping 20-page-long online User Guide.

Ziru User Guide

Electronic copy is available online by following this link:

www.dodrotu.com/ziru/user-guide/4.4.822.pdf

