

# Ziru: A Very Quick Start

## Turn On

- Press Power button once;

OR, failing that:

- Press and hold Power button for 5 seconds.

Ziru also turns on automatically when you disconnect the power cable.

Power button is located at the bottom right corner of Ziru.

## Voice Commands

- Next
- Previous
- Zoom In
- Zoom Out
- Display Battery
- Read Text
- Brightness Up
- Brightness Down
- Flash On
- Flash Off
- Reset Zoom
- Volume Up
- Volume Down
- Yellow Outlines
- Black Background

## Turn Off


- Press Power button once.

## Charge

- Insert USB cable into charger;
- Plug in Charger into wall socket;
- Insert the other end of the cable into the charging port.

Charging port is located on the left hand side in the centre.

# Customer Support Dodrotu Limited

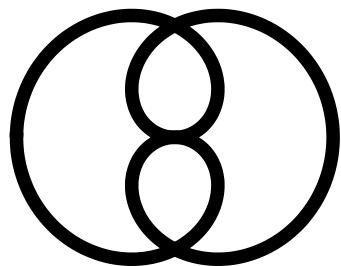

Email:  
support@dodrotu.com

Phone:  
+44 07577 669 499

Web:  
www.dodrotu.com/ziru

## About Your Ziru

Version: 4.6.828

Build date:  
2024-01-27

Variant:  
Enter Glass Dragon

## Health and Safety Warning

This variant of Ziru has a number of limitations.

By using the product you agree to read and abide by the Product Limitations and Health and Safety Warnings sections of Ziru User Guide.

## Further Info

Full functionality is described in a gripping 20-page-long online User Guide.

## Ziru User Guide

Electronic copy is available online by following this link:

[www.dodrotu.com/ziru/user-guide/4.6.828.pdf](http://www.dodrotu.com/ziru/user-guide/4.6.828.pdf)

